# Saljol

## Wohnraumrollator Page

Gebrauchsanleitung | User manual

# 1. Vorwort

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## SALJOL Indoor rollator Page User manual

# 1. Introduction

Thank you for choosing to purchase a residential rollator from Saljol! With your rollator you can lead a mobile and self-determined life with style. The living room rollator Page focuses on safety and will support you in your everyday life.

Please read the instructions for use carefully before using your Page indoor rollator for the first time. It contains important safety instructions, useful tips for use and care, and a list of recommended accessories.

## Table of contents

1. Introduction.....	26
Table of contents.....	26
2. Identification.....	28
3. General notes.....	29
4. Scope of delivery and product overview.....	31
4.1 Scope of delivery.....	31
4.2 Marking.....	31
4.3 Product overview.....	32
4.4 Safety Instructions.....	34
5. Setting up the living room rollator.....	34
5.1 Correct frame size.....	34
5.2 Insert seat board.....	35

6. Use of the rollator.....	35
6.1 Grip and support positions when walking and standing within the Page.....	35
6.2 Grip and support positions when walking and standing outside the Page.....	36
6.3 Standing up and sitting down.....	36
6.4 Braking with foot brake.....	36
6.5 Brake and parking brake.....	37
6.6 Determining the direction.....	39
6.7 Moving on the rollator while sitting.....	39
7. Accessories.....	40
8. Cleaning and disinfection.....	43
8.1 Cleaning.....	43
8.2 Disinfection.....	44
8.3 Maintenance.....	45
9. Storage and shipping.....	45
9.1 Storage conditions.....	45
10. Disposal.....	45
11. Technical data.....	46
12. Warranty.....	47
13. Test report.....	48

## 2. Identification

Manufacturer: Saljol GmbH

Product: Indoor rollator

Type: Indoor rollator, serving trolley and commode chair

Article descriptions: Page Indoor rollator Version: 2020

Manufacturer:

Saljol GmbH

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82216 Maisach

Phone: +49 8141 317740

Fax: +49 8141 3177411

hallo@saljol.de www.saljol.de

Formalities of the operating instructions:

Version/Revision: 2021.07

Date of creation: 06.07.2021

Information corresponds to the status at time of printing.

Subject to technical changes. Technical drawings and photos may differ from reality.

© Copyright Saljol 2021

All rights reserved.

Reprinting, even in extracts, only with the permission of Saljol GmbH permitted.

## 3. General notes

### Purpose:

The living room rollator Page from Saljol is designed for indoor use and smooth floors. It supports people with balance or gait problems and contributes to increased mobility of disabled people. On the seat board the user can pause and move him- or herself around in a seated position.

### Indication:

People who need additional stability support while walking and who have sufficient walking ability, safety and strength to move around at home with the help of the Page. The Page supports movement-impaired people when getting up, sitting down and transporting. The seat board enables people to rest and move around tripping on flat, even floors.

### Contraindication:

The Page is not for people with weak arm strength, significant cognitive limitations or very poor balance suitable. The use and application possibilities of the Page, should be evaluated and instructed by a doctor or therapist.

### Useful life:

When used as intended, regular care and maintenance by a specialized company and compliance with the safety instructions, the expected service life of the living room rollator is up to 4 years. If after 4 years the rollator is still in proper condition, it can be used again.

### Responsibility:

We can only consider ourselves responsible as manufacturer for the safety, reliability and usability of the living room rollator, if changes, extensions, repairs and maintenance work are carried out regularly and only by authorized specialists and if the rollator is used in accordance with all instructions for use.

### Declaration of conformity:

Saljol GmbH as manufacturer of the product declares that the Page in all color, brake and size variations meets the requirements of the European Parliament and Council Regulation 2017/745 of the harmonized Standards EN ISO 11199-2:2005 – Walking aids for double-armed use – Requirements and test methods – Part 2: Rollators ISO 11199-3:2005 – Walking aids for double-armed use – Requirements and test methods – Part 3: Walker corresponds.

## 4. Scope of delivery and product overview

### 4.1 Scope of delivery

1x Page indoor rollator  
1x seat board  
1x instruction manual

### 4.2 Marking

The serial number for further information is located on the bottom of the frame.

## 4.3 Product Overview

The living room rollator Page

### 1 Frame

Secure support from all sides. Serves as mobile handrail.

### 2 Wooden seat

Move through your apartment while sitting.

### 3 Hand brake (only for model with handbrake)

### 4 360° rotatable wheels

Rotates on the spot. Easy handling in narrow corridors. In the version with hand brake the rear wheels are lockable in longitudinal direction. For the version with foot brake, the parking brake is located on the rear wheels.

Available in the sizes S, M, L and XXL

## 4.4 Safety instructions

The safety instructions are marked twice:  
Through a symbol and by a signal word.

### Important!

In these boxes you will find application tips, useful information and obligations for a special behaviour or activity for the safe use of the indoor rollator Page.


## 5. Setting up the living room rollator

### 5.1 Correct frame size

The recommended size of the rollator can be found in the following table.

Size S	Size S for a body height of 145–165 cm
Size M	Size M for a body height of 150–180 cm
Size L	Size L for a body height of 170–200 cm
Size XXL	Size XXL for a body height of 170–210 cm from a hip width of 45 cm.

## 5.2 Insert seat board

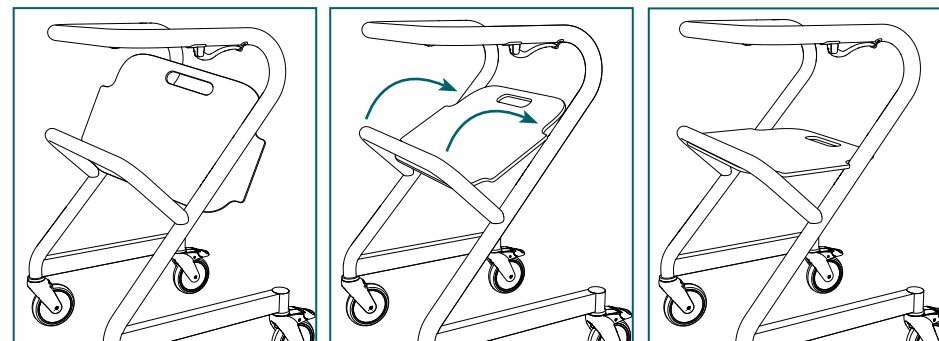Check that the seat board is firmly seated before you sit down.

## 6. Use of the rollator

### 6.1 Grip and support positions when walking and standing within the page

## 6.2 Grip and support positions when walking and standing outside the page

## 6.3 Standing up and sitting down

When standing up and sitting down, the parking brake must be activated.


## 6.4 Braking with foot brake (for the page version with foot brake)

Press foot pedal down for locking.

The wheel is blocked. Press inclined pedal to release.

## 6.5 Brake and parking brake (for the Page version with handbrake)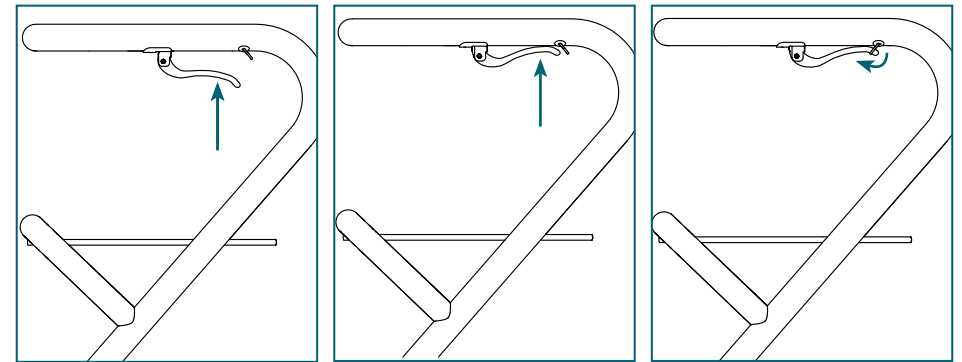

**Brake:** Pull up the brake handles on both sides.

**Parking brake:**  
1. Pull the brake handle upwards.  
2. Fold the bracket to the front.


### Important!

When you sit on the living room rollator to rest, it is absolutely necessary to lock the parking brake.

Adjustment and control of the hand brake:

1. Loosen lock nut "1"
2. Unscrew threaded sleeve "2" (= brake becomes tighter) or screw in (= brake becomes smoother)
3. Check braking force
4. Repeat point 2-3 until desired setting is found
5. Tighten lock nut "1"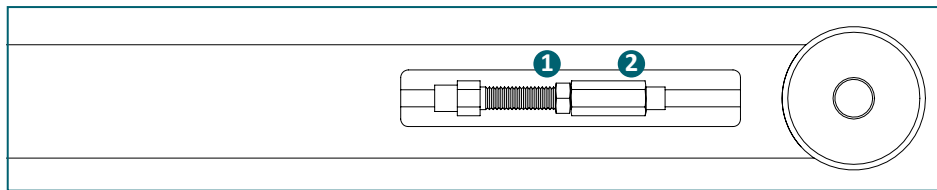

### Important!


The brake is correctly adjusted if the rear wheels lock when the brake is applied and do not turn when pushing.

### Info

Here you can find a video, where the brake adjustment is explained:  
[www.saljol.de/en/technical-data-living-room-rollator/](http://www.saljol.de/en/technical-data-living-room-rollator/)

## 6.6 Locking the direction (Page version with handbrake)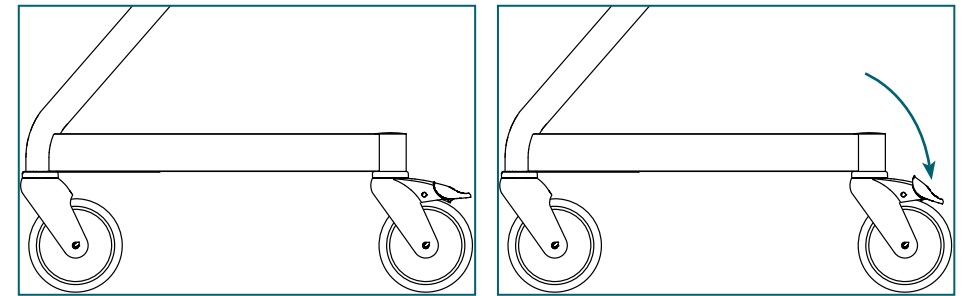

Press foot pedal down to lock the directional locking of the rear wheels.

## 6.7 Moving on the rollator while sitting

It is allowed to walk around the apartment with the living room rollator Page, while seated

1. Check that the seat board is firmly held
2. Activate parking brake
3. Sit down
4. Release parking brake

Now you can move through the apartment while sitting

## 7. Accessories

### Note

The following accessories are available for the living room rollator in sizes S, M and L. No separate accessories are available for size XXL.

### Tray

The stable wooden tray can be loaded up to 20 kg and has a large usable surface. The tray turns the Page into a serving trolley with which you can even drive over the table and safely put the tray down - without danger of crushing your fingers.

### Back roll

The back roll can be attached to the frame of the living room rollator with a zipper. This makes the rollator a comfortable tripple chair. It can also be used as impact protection on the side tubes or the crossbar. Also available in synthetic leather dark brown.

### Upholstered seat

With the upholstered seat your living room rollator becomes a comfortable tripple and transfer wheelchair. Ideal for everyone who uses the rollator, e.g. as a kitchen or work chair. The seat has a load capacity of up to 120 kg and is easy to attach and detach. Also available in synthetic leather dark brown.

### Wicker basket

The hand-woven wicker basket can be loaded with up to 20 kg and can be used for transportation.

Mounted at the top

Mounted below

### Cabinet

The cabinet can be loaded with 20 kg and can be simply hung up in the frame of the living room rollator. With its two storage compartments, the cabinet is ideal as a bedside table.

### Bag

The bag made of medium grey fabric can be loaded with 20 kg and is ideal for transportation.

Mounted at the top

Mounted below

### Ergo seat

The ergonomically shaped seat adapts to the body contours and is covered with brown synthetic leather.

### Seat cover

The seat cover is simply pulled over the seat board and secured with Velcro. The grip hole remains unchanged. Slipping away while sitting is reduced by the seat cover.

### HPL seat board

The HPL seat board is wipe disinfectable and antibacterial. As the surface does not provide a breeding ground for bacteria, the seat board is hygienic.

### Note

You can order the accessories in our online store [www.saljol.de](http://www.saljol.de) around the clock or by phone through our customer service (Mon. – Fri. 9 am – 16 pm) at +49 (0) 8141 317740 conveniently from home.

## 8. Cleaning and disinfection

### 8.1 Cleaning

#### Frame:

Cleaning can be done with warm water and a sponge. For stubborn dirt a commercial cleaning agent can be added to the water. Disinfectants can be used without problems. Plastic parts are best maintained with a commercial plastic cleaner. Please observe the instructions for use of the cleaning agent used.

**Important:**

Never use aggressive cleaning agents such as solvent / abrasive or hard brushes.

**Wheels:**

Clean the wheels with warm water and a mild solvent. If necessary, use a plastic brush, never a wire brush.

**Seat board:**

Clean the seat board with a cloth and warm water. If necessary, use a mild detergent. Dry the rollator thoroughly with a cloth after cleaning.

**Important**

Never clean the rollator with a high pressure / steam cleaner!

**8.2 Disinfection**

The living room rollator can be disinfected with household disinfectants. The Robert Koch list of approved disinfectants can be found at [www.rki.de/EN/](http://www.rki.de/EN/). Let the rollator dry in the air and note the exposure time of the disinfectant you use.

**8.3 Maintenance**

We recommend the Page living room rollator at least once per year, ideally even twice, an inspection by the specialist dealer. If faults or defects occur during use, the indoor rollator must be returned to the dealer immediately for repair. Saljol reserves the right to limit the warranty claims if the walker has not been regularly maintained.

**9. Storage and shipping**

If the Page needs to be stored or shipped, it is advisable to keep the original packaging and store it for these purposes so that it is available when needed. Then your rollator is optimally protected against environmental influences during storage or transport.

**9.1 Storage conditions**

The rollator should be stored in dry, preferably temperature-controlled rooms, protected from light, in such a way that no pressure load is applied from the outside.

Temperature: -10° to +35°C

Humidity: 45% to 60%

**10. Disposal**

Dispose of the materials separately according to type in the designated collection systems. Please dispose of special waste in the collection systems provided.

## 11. Technical data

Property:	S	M	L	XXL
Handle height	80 cm	86 cm	92 cm	95 cm
Seat height	52 cm	55 cm	58 cm	58 cm
Recommended body height	135–165 cm	150–180 cm	170–200 cm	170–210 cm from hip width 45 cm
Max. user weight	150 kg	150 kg	150 kg	200 kg
Largest length	51 cm	51 cm	51 cm	75 cm
Largest width (bottom)	56 cm	56 cm	56 cm	68 cm
Largest width (top)	51 cm	51 cm	51 cm	65 cm
Distance between the handles	43 cm	43 cm	43 cm	60 cm
Weight with handbrake	13,5 kg	14,0 kg	14,2 kg	16,4 kg
Colour	white anthracite	white anthracite	white anthracite	anthracite
Foot brake	yes	yes	yes	yes
Handbrake	yes (depending on model)	yes (depending on model)	yes (depending on model)	yes

## 12. Warranty

Saljol gives a 2-year warranty against production or material defects.

Warranty services refer to all defects of the product, which can be proved to be defective in material or manufacturing are to be attributed.

You can register the residential rollator online to extend your warranty to 4 years from the date of purchase. The Registration must be received no later than 14 days after date of purchase take place. The registration takes place under [www.saljol.de/en/extended-warranty/](http://www.saljol.de/en/extended-warranty/)

All claims for warranty and guarantee expire, as soon as the residential rollator has been deliberately destroyed, this manual was not observed, the Living room rollator improperly and from an untrained person has been serviced or repaired or if the living room rollator was operated with non-original accessories. These warranty conditions only apply to products purchased in Germany. The specialist dealer will provide information on the warranty conditions for products purchased abroad.

# 13. Test report

## Questions:

- Does the product have all labels attached?
- Hand brake version: Do the hand brakes work?
- Hand brake version: Does the directional locking of the wheels work?
- Foot brake version: Do the foot brakes work?
- Are the brake handles firm and do not wobble?
- Are the maintenance flaps of the brake closed and firm?

Date	Inspector	Stamp / Signature
Date	Inspector	Stamp / Signature
Date	Inspector	Stamp / Signature
Date	Inspector	Stamp / Signature
Date	Inspector	Stamp / Signature