
Bedienungsanleitung
User manual
Manual de instrucciones
Mode d'emploi

DH-40 R

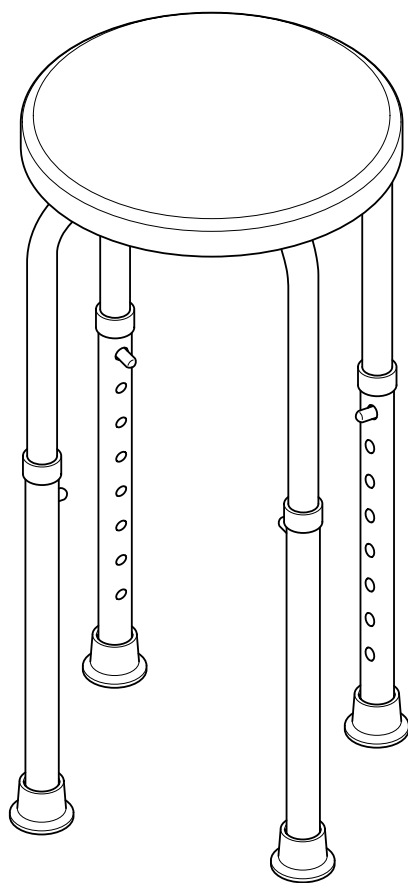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Notizen

Contents

1. Introduction	12	3.2. Product image/scope of delivery	16
2. Safety information	13	3.3. Product's identification label	16
2.1. Labels and symbols	13	4. Assembling the shower aid	17
2.2. Intended use	13	5. Usage and care	18
2.3. Indications	13	6. Maintenance and repair	18
2.4. Contraindications	13	7. Transfer/Re-use	19
2.5. General safety information	14	8. Storage and shipping	19
2.6. Special safety information	14	9. Disposal	19
2.7. Service life	15	10. Warranty	19
2.8. Declaration of compliance	15	10.1. Warranty certificate	20
3. Components/Overview	15	Explanation of symbols	A1
3.1. Technical data	15		

1. Introduction

Dear Customer,

You have chosen a high-quality shower aid from Bischoff & Bischoff.

Thank you for trusting in us.

This User manual contains all the necessary information for assembling, using and looking after the DH-40 R shower aid.

Before using your new shower aid for the first time, please read and follow the information in this User manual carefully, including all safety information.

This manual is an integral and necessary part of the product. Keep the User manual within easy reach and pass it on with the shower aid when transferring the product.

For individuals who are visually impaired, this document is available as a pdf file on our website www.bischoff-bischoff.com.

2. Safety information

2.1. Labels and symbols

IMPORTANT!

Indicates particularly useful information in a particular context.

You will find the explanations of any additional symbols used in the appendix at the end of this User manual.

2.2. Intended use

A shower stool supports independent showering when physical restrictions which make it difficult to stand safely in the shower.

2.3. Indications

Use of the shower aid is advisable if you have difficulty standing up or remaining standing due to

- Damage to the functionality of the lower extremities with limited or failing mobility and muscle strength (e.g. loss of strength through malignant diseases, chronic, advanced cardiovascular and pulmonary diseases) and/or
- Damage to the vestibular functions or tendency to fall due to different genesis with, at the same time, sufficiently stable trunk posture and sufficient residual functionality of the upper extremities.

2.4. Contraindications

Use of the shower stool is not advisable for individuals with:

- Perceptual disorders,
- Balance disorders,
- No capability to sit,

IMPORTANT!

Despite thorough research and editing, incorrect information may occur in our User manual.

We reserve the right to make technical modifications.


You can find the current version of the User manuals in the download area of our website www.bischoff-bischoff.com.

2.5. General safety information

Notification of incidents: The provider or user must report

1. any malfunction,
 2. any change in the characteristics or performance as well as
 3. any inadequacy in the labelling or instructions for use of a medical product, which has led or could lead to death
- or a severe deterioration in the health condition of a patient, an employee or a third party, must be reported immediately to the national authority responsible for medical products.

2.6. Special safety information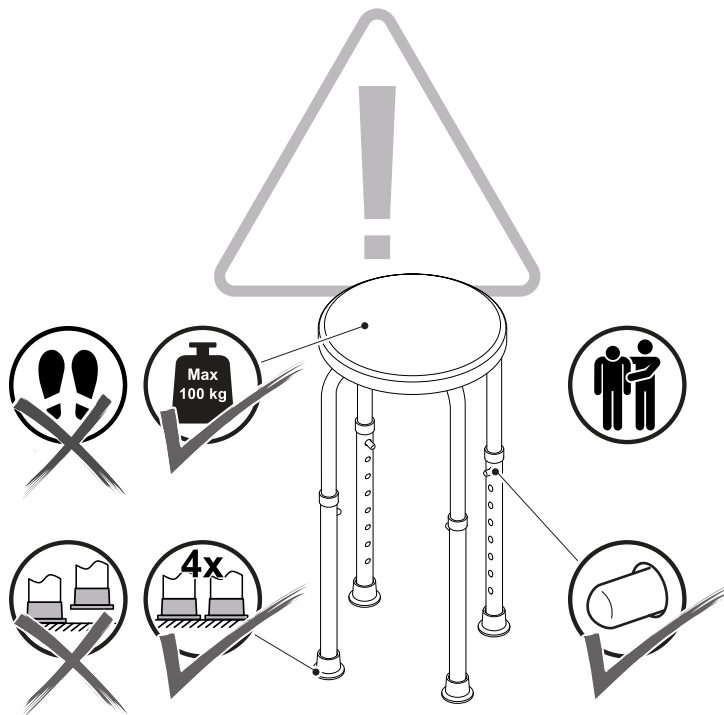

IMPORTANT!

You will find the description of any symbols used in the appendix at the end of this User manual.

2.7. Service life

The expected service life is up to 5 years, if the device is used correctly, and safely, following the care and maintenance instructions.

Usage beyond this period is at own risk and is only allowed if the product is in a condition allowing its safe usage.

2.8. Declaration of compliance

B+B as the product's manufacturer declares that the DH-40 R shower stool conforms to the requirements of the

EU Regulation 2017/745 for medical products as well as the 93/42/EEC Directive for medical products.

3. Components/Overview

3.1. Technical data

Seat width (diameter):	330 mm
Seat height adjustment:	390 - 570 mm
Unladen weight:	1.2 kg
Max. Payload:	100 kg
Colour of seat:	white
Seat material:	HDPE
Frame material:	Aluminium

3.2. Product image/scope of delivery

At receiving the goods, please check that all components have been delivered. The package should contain the following components (Fig. 1 and 2):

- ① Seat (1x)
- ② Telescopic tubes (2x)
- ③ Foot sections (4x)
- ④ Rubber feet (4x)
- ⑤ Wing screw (5x)
- ⑥ Washer (5x)
- User manual (1x)
- Packing

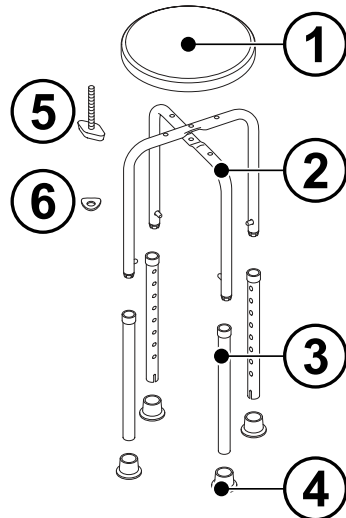
Fig. 1

3.3. Product's identification label

The product's identification label (Fig. 3) is located on one of the telescopic tubes ②.

- Ⓐ Manufacturer's logo
- Ⓑ Manufacturer information
- Ⓒ Date of manufacture
- Ⓓ Model name
- Ⓔ Catalogue number
- Ⓕ Medical product in compliance with EU Regulation 2017/745
- Ⓖ CE mark
- Ⓗ Read the User manual!
- Ⓘ Maximum payload
- Ⓙ Unique Device Identification (UDI)
- Ⓚ Serial number





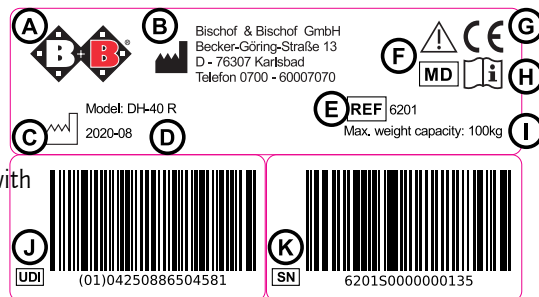
Fig. 2

4. Assembling the shower aid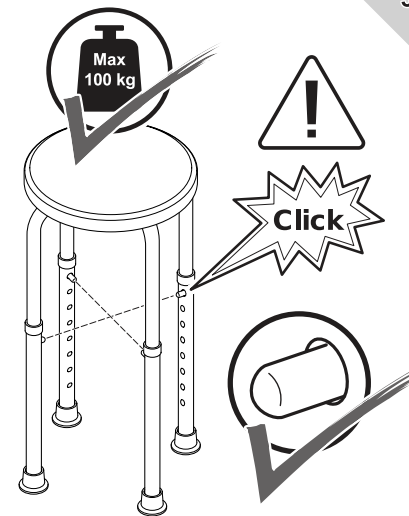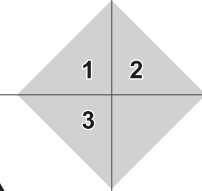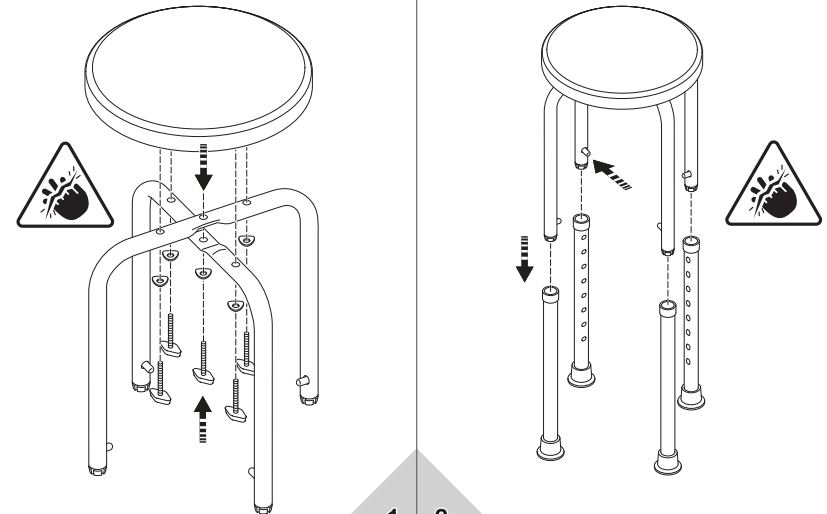

IMPORTANT!
Disassemble the shower aid by following the instructions in reverse order.

5. Usage and care

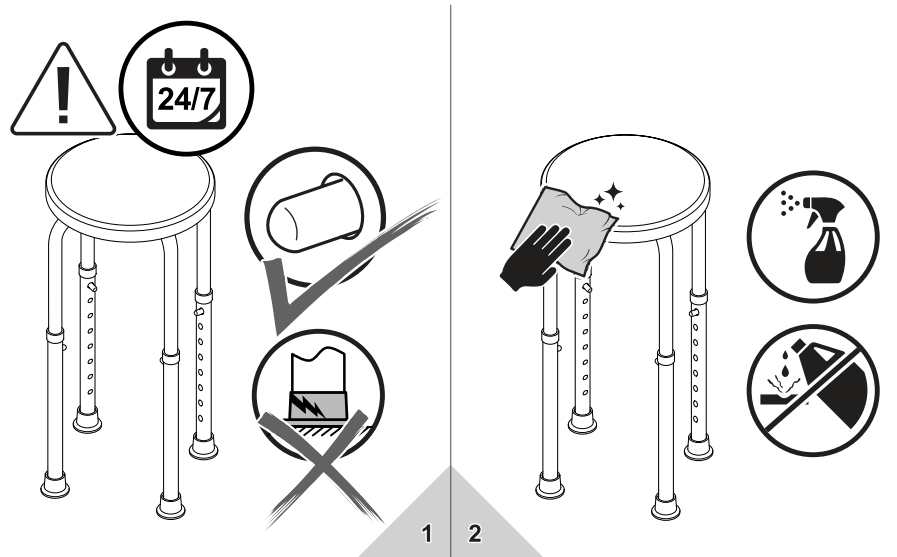6. Maintenance and repair

Your shower aid is made from high-quality materials for long-term, comfortable use. We carry out in-depth testing of all our products before they leave our factory. However if you should find that your shower aid has a fault, you should no longer use it. Contact the healthcare professional who supplied the product to you to have the product repaired. He will be happy to help you.

Furthermore, we recommend that the shower aid is serviced at least once a year. Your

dealer will receive a maintenance plan from us together with further documents which are necessary to carry out an inspection.

Only authorised partners (medical supply stores or dealers) are permitted to carry out repairs and maintenance work.

If you have any questions about maintenance and repair, our Customer Service will be happy to help on the free phone number +49 (0)700 6000 7070.

7. Transfer/Re-use

Your shower aid is designed for re-use. Before passing the shower aid to another user, it needs to be serviced and disinfected.

When the shower aid is passed to another user, please also hand over this User manual and the address of your dealer.

8. Storage and shipping

Before you store or ship the product, remove all parts which have been plugged in and are not fixed parts. First, pack these individually and then pack them all together in a suitable cardboard box.

Where possible, retain and use the original packaging for this purpose. In this way the product has the highest level of protection against environmental influences during storage or transportation.

9. Disposal

Please contact your dealer for professional disposal of the product.

If you would like to dispose of the product yourself, please observe the local municipal regulations!

10. Warranty

The warranty covers all defects which are proven to be due to faulty material or manufacturing. In the event of a claim, the fully completed warranty certificate must be submitted with the product. B+B provides the legal warranty of 2 years.

Failure to observe the instructions in the User manual as well as not carrying out maintenance work properly and/or making technical modifications and additions (attachments) without the agreement of Bischoff & Bischoff GmbH will lead to loss of warranty and the manufacturer's liability.

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B+B France S.A.R.L.