Toilettenrollstuhl „TRS 130“
Wheeled Drop-Arm Commode

Art.Nr. / REF: **530 100 000**

CE

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Dear users,

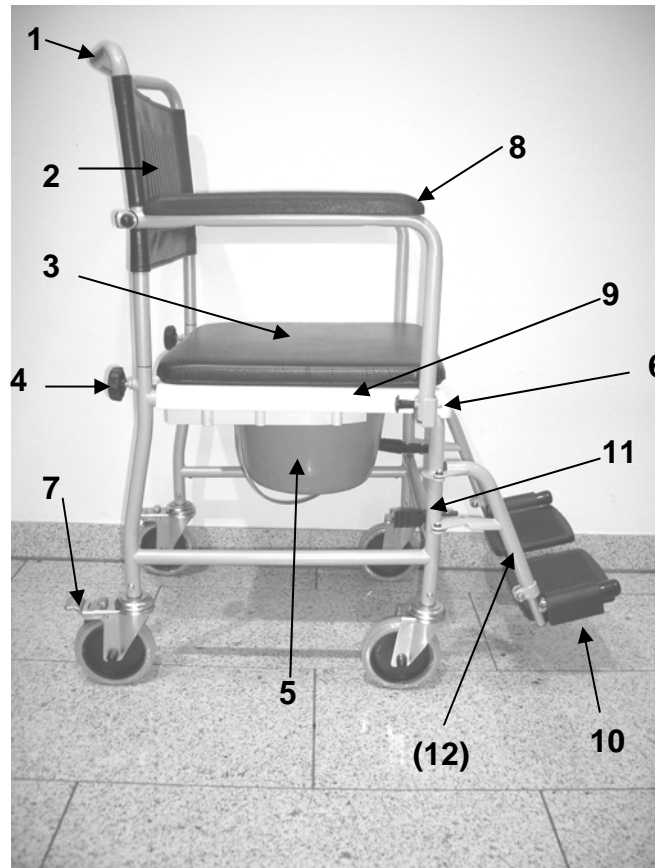
we congratulate on the buying of a technically matured and the best available technology product.

This instructions helps you and the carers as a basis for sure, safe use of the toilet wheel chair and a sensitive use of the product's possibilities.

The instructions describes the preparation, service, use, servicing and cleaning of the product.

This instructions is to be perused carefully and must be always accessible to all users of the product.

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Toilet wheel chairs are necessary when using or achievement of the toilet is not possible because of a handicap. Toilet wheel chairs are also usable as room wheel chairs. Toilet wheel chairs can be pushed over customary toilet bowls.

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1. *Wheeled Drop-Arm Commode*
2. *Bucket with cover*
3. *Footrest*
4. *Heel tape*
5. *Operating instructions*

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*This product is suitable for the reapplication.
With the processing the standards of the manufacturer must be considered.
This standards are made available on inquiry.*

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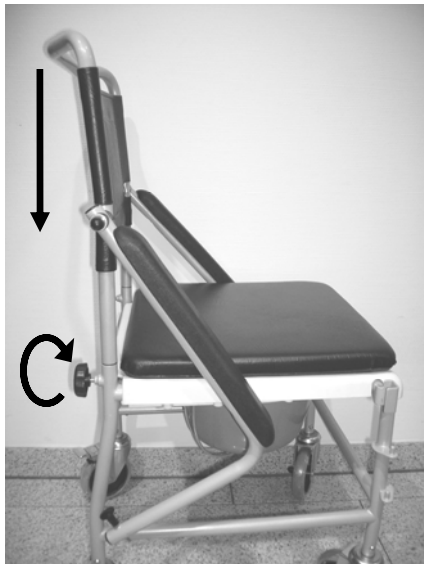
The Wheeled Drop-Arm Commode is mountable nearly without tools

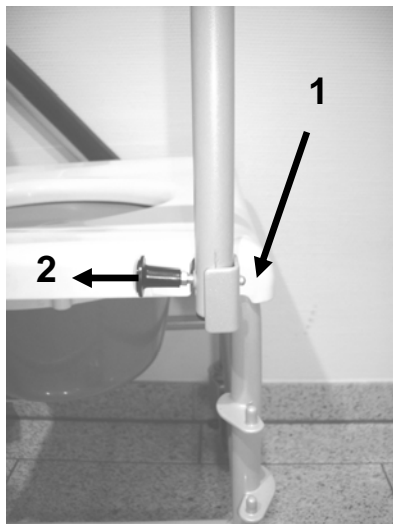
Put the premounted backrest on the basic rack and protect with the star screws with a clockwise rotation.

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*Pull both armrests upwards until they **audible** engage the interlock. The interlock occurs automatically. Make sure that the interlock is successful on both sides, on this occasion, the locking pin (1) must be visible. In order to unlock the armrests move the locking button to the back (2) and swing the armrest downwards.*

ATTENTION: *It may be possible that the armrest is a little bit bent during transport, this can be postbent easily until the locking button in front properly locks.*

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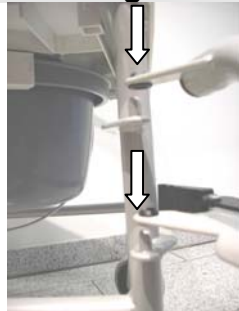
*To interlock the brake press lever downwards. **This will lock the direction of the wheel in place.***

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
Hang up the provided footrests with both drillings as shown in the respective hold pins

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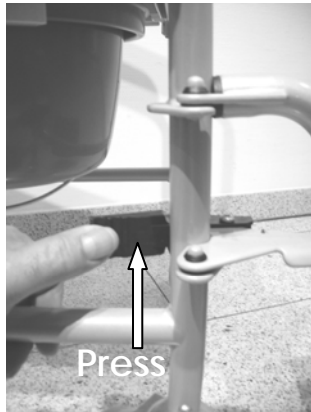In order to interlock the footrests turn them to the middle until its audible engage.

In order to unlock press the lever as shown inwards.

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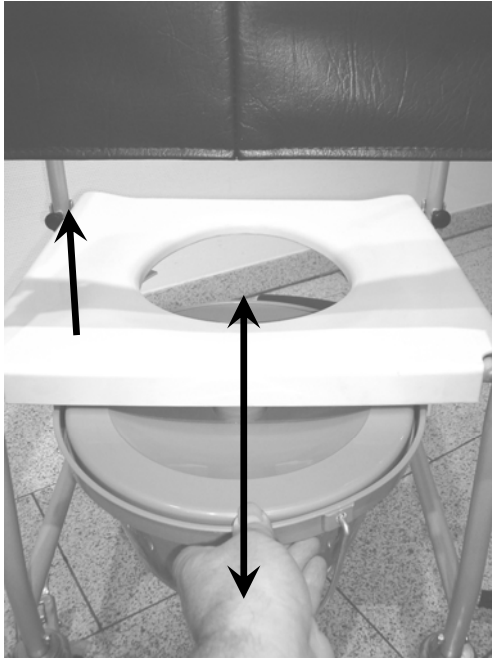
To adjust the footrests in the height first unlock the fixing clamp (1). After pinching the fixing bolts (2) the footrests can be shifted (3) in 4 steps in the height for leg lengths from 420 – 500 mm.

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Insert or remove the toilet bucket (also with put on lid) always from the back. The toilet bucket engages, the toilet seat can be taken simply upwards for cleaning.

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Clean regularly by hand with a damp cloth and usual household cleaning agents!

Use spraying disinfectants p.e. Descosept AF or Bacillol AF.

Do not use any aggressive detergents!

Screws of TRS 130 must be checked regularly and if necessary retightened.

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Frame: Welded steel tube, chrome plated

Sitzbrille: PP (Polypropylen)

Toilette seat: PP (Polypropylene)

Armauflagen: PU (Polyurethan)

Armrests: PU (Polyurethane)

Typenschild – Type label

The image shows a rectangular type label for a toilet roll chair. On the left side, there are three numbered arrows pointing to specific fields: 1 points to the product name, 2 points to the reference number, and 3 points to the serial number and date of production. On the right side, there is a fourth arrow pointing to the CE mark. The label contains the following text: 'Toilettenrollstuhl' in a large font, 'drive' in a blue logo with 'MEDICAL DESIGN & MANUFACTURING' underneath, 'TRS 130' in a large font, 'REF 530 100 000 V.4' in a box, 'SN / MMYX XXX XXX / 46/10' in a box, 'Max. Belastung - Max.load: 130 kg' in a bold font, and 'Drive Medical GmbH & Co. KG 88316 Isny/Allgäu (Germany)' at the bottom left. The CE mark is at the bottom right.

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We issue a warranty of 24 months from date of purchase.

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Subject to technical alterations. Errors and omissions excepted.

Stand - *as per*: **10/2010**

**Drive Medical GmbH & Co. KG
Leutkircher Straße 44
D-88316 Isny / Allgäu (Germany)**

**Drive Medical Limited
Ainleys Industrial Estate
Elland
GB WEST YORKSHIRE HX5 9JP**

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