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**drive**  
MEDICAL DESIGN & MANUFACTURING

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*Dear users,*

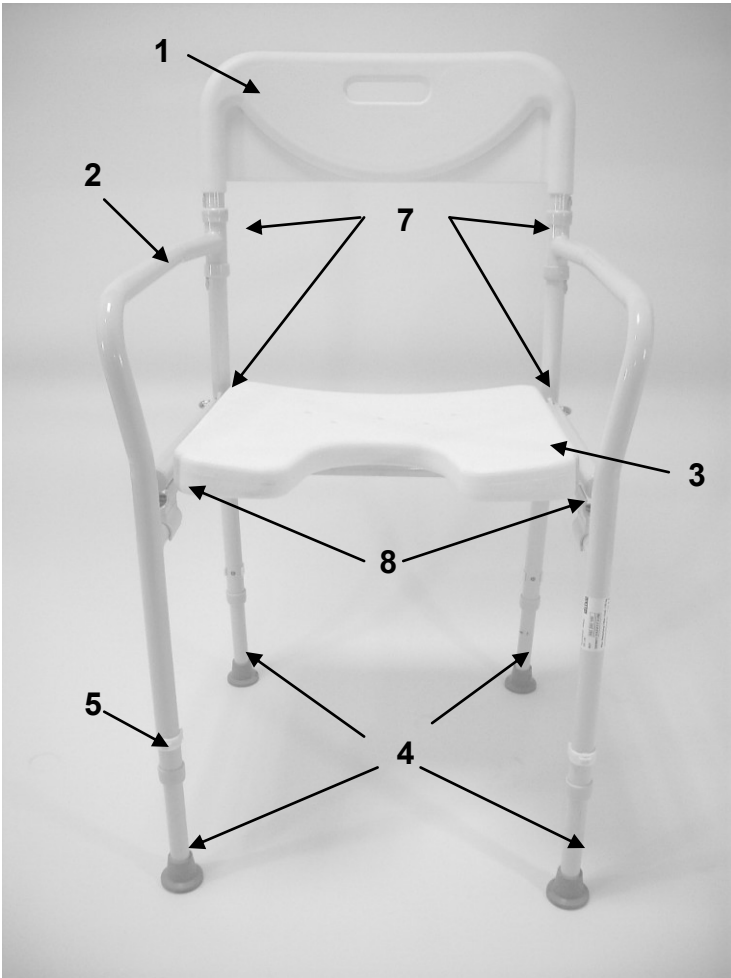
*we congratulate on the buying of a technically matured and the best available technology product.*

*This instructions helps you and the careers as a basis for sure, safe use of the Shower chair and a sensible use of the product possibilities.*

*The instructions describes the preparation, service, use, servicing and cleaning of the product.*

*This instructions is to be perused carefully and must be always accessible to all users of the product.*

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*Standing insecurity / standing inability with use decrease / of missing functional ability of the lower extremities or other illnesses who are affecting the stability (e.g., strength loss with evil illnesses, chronic, advanced heart circulatory illnesses and lung illnesses) infers with a body position weakness, so that an independent use of the shower is still allowed.*

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*This product is suitable for reapplication.*

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*With the processing the standards of the manufacturer must be considered. This standards are made available on inquiry.*

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
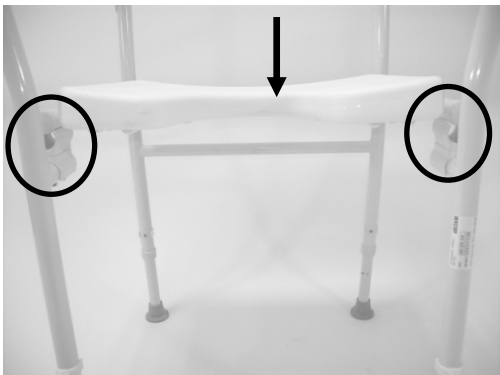
*The shower chair is completely assembled.. Unfold him as illustrated and put the seat upwards.*

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*Swing both side parts forwards and then the seat record down them on the hold clips lies (picture 1).*

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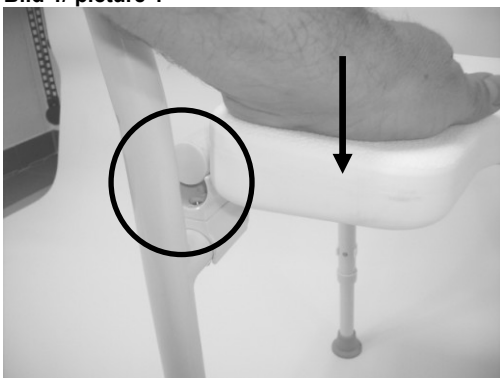*Press now the seat plate hard down until it engages audible in the hold clips on the left and on the right side (picture 2).*

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

*Check the correct position of the seat plate in the hold clips. (Picture 3)*

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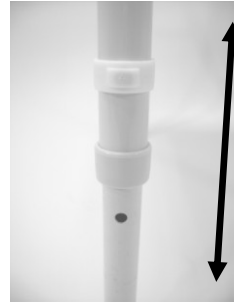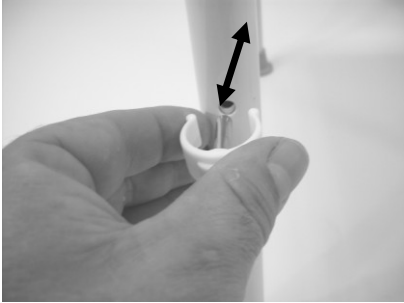*To adjust the footrests in the height first unlock the fixing clamp. Then the the footrests can be shifted in 4 steps in the height for seat heights from 460 – 530 mm.*

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*Clean regularly by hand with a damp cloth and usual householf clening agents!*

*Use spraying disinfectants p.e. Descosept AF or Bacillol AF.*

*Do not use any aggressive detergents!*

*Screws of DS 130 must be checked regularly and if necessary retightened.*

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**Frame:** Welded steel tube, powder coated  
**Seat plate and backrest:** PE-HD

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*We issue a warranty of 24 months from date of purchase.*

*Subject to technical alterations.  
Errors and omissions excepted.*

**Drive Medical Limited**  
**Ainleys Industrial Estate**  
**Elland**  
**GB WEST YORKSHIRE HX5 9JP**

Internet [www.drivemedical.co.uk](http://www.drivemedical.co.uk)

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Drive Medical GmbH & Co. KG 88316 Isny/Allgäu (Germany)
**Drive Medical GmbH & Co. KG**