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Operating instructions  
Hanging bath seat "BOB"  
adjustable

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Bath seats are hung up in the bath tub. They have to be mastered by own strength and the lowering is thereby decreased.

They are considered among other things when considerable functional restrictions of the upper and/or lower extremity are. Nevertheless, with application of the aid bathing can be done again widely independently. However, the application assumes that a rest effectiveness enough exists with regard to the body posture and the use of the extremities for the coping with position changes still

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### Content:

Seat plate with frame  
Hang up device left  
Hang up device right  
Backrest (optionally)

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Adjust the hang up devices on the left and on the right in the guiding pipe of the frame until your desired width have reached, press moreover the respective locking pin.

Pay attention that the locking pins have engaged in the final position !

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*The tub edge must be clean and dry. Adjust the hang up devices always on maximum width to guarantee a safe support.*

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*The backrest is mounted ready and must be inserted in the admission pipes (1) at the underside of the seat plate. The locking pins must engage in the corresponding holes. (2)*

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*Do not use the Bath Chair if it is damaged!*

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*Danger of falling ! Supporting surface must be dry and wide enough !*

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*Product must not be used when surrounding temperature is over 60°C (e.g. use in a sauna etc.)*

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*For cleaning and disinfection use commercial products*

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*The bath seat is suitable for further use after technical check by a specialized dealer.*

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*We issue a warranty of 24 months from date of purchase.*

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